Instructions for Use

- Please read the instructions carefully before use.
- Remove all packaging before use.
- Place the highest point of the wedge under the knees with the short side under the patient's thighs.
- The Softform Odstock Wedge is designed to be used in conjunction with the Softform Heelpad to provide improved support of the lower limbs.

Product Description

The Invacare Softform Odstock Wedge reduces pressure under the heels without increasing pressure on the thighs and calves. It improves and maintains posture in the semi-recumbent position; the support under the thigh reduces the tendency for patients to slip down the bed, thus reducing shear and friction forces on the sacral area. By tilting the pelvis, it also reduces lumbar lordosis - desirable following back surgery and ideal for improving the sleeping position of people with back problems. Available in the following size:

Odstock wedge	Length	Width	Height
	86.5cm	57cm	11.5cm

If you require more information on the Softform Accessory range please telephone the customer services department at Invacare on +44 (0) 1656 776222 who will be happy to help you.

Repositioning

It is very important for the patient to reposition themselves, or to be repositioned, regular basis. This relieves pressure; helps prevent both tissue compression and potential ulcer formation.

Cleaning, Disinfection and Re-processing

- The cleaning protocols should be undertaken as part of a cleaning regime or in event of a change of user. The cover can be wiped down using a solution of new detergent and hot water, applied with a single-use wipe. Cleaning agents contain phenols or alcohols should not be used.
- In the event of the cover becoming contaminated, please ask your Infection Control Officer for advice or alternatively, to disinfect the cover apply a 0.1% chloride solution with a single use wipe and rinse with clean water.

Changing the cover

The cover cannot be changed as it is welded to prevent fluid ingress.

Caution

Minimum user weight: 25Kg

Maximum user weight: 249Kg

Quality and Flame Retardancy

Quality is fundamental to the company's operation, working within the disciplines ISO 9001:2000 and ISO 13485:2003. All Invacare products feature the CE mark compliance with the Medical Devices Directive.

The foam used to manufacture the Invacare Softform Odstock Wedge is independently tested and certified in accordance with BS 5852:2006 ignition source 0, 1 and crib 5. The cover is tested to BS7175:1989.

Manufactured by Invacare UK Ops Ltd
Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ UK
Tel: +44 (0) 1656 776222 Fax: +44 (0) 1656 776220

In keeping with our policy of continuous improvement, we reserve the right to modify design without prior notice.

Softform is a registered trademark of Invacare LTD.

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Minimum gebruikersgewicht: 25Kg

Maximum gebruikersgewicht: 249Kg

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Quality is fundamental to the company's operation, working within the disciplines of ISO 9001:2000 and ISO 13485:2003. All Invacare products feature the CE mark, in compliance with the Medical Devices Directive.

The foam used to manufacture the Invacare Softform Odstock Wedge is independently tested and certified in accordance with BS 5852:2006 ignition sources 0, 1 and crib 5. The cover is tested to BS7175:1989.

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Manufactured by Invacare Ltd