**6358E
9680E - 9681E
H203
H207 - H209 (+O100)**

F	1.	Guide utilisateur
GB	2.	User's manual
D	3.	Gebrauchsanweisung
E	4.	Manual del usuario
I	5.	Manuale dell'utente
NL	6.	Gebruiksaanwijzing
P	7.	Guia do utilizador
S	8.	Bruksanvisning
DK	9.	Brugsanvisning
FIN	10.	Käyttäjän käsikirja
N	11.	Bruksanvisning

I. FRANCAIS

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2. ENGLISH

You have just purchased an INVACARE® product. Our products are designed and manufactured to the most rigorous quality and safety standards and conform to the European directives (MDD 93/42/EEC). They are guaranteed for one year. Certain products may have a longer guarantee; please check with your dealer.

The following operating and maintenance instructions must be observed:

- Check that the product has been delivered in perfect condition before you use it.
- If you have any doubts about the restrictions on use, the adjustment or maintenance of the product, please contact your dealer.
- Do not exceed the maximum operating load or the maximum load shown on the product label.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Worn, damaged or missing components must be replaced immediately by an approved repairer.
- Do not leave or store the product out of doors.
- Periodically check the stability and rigidity of the product and, where applicable, that the screws are correctly tightened and the rubber end caps in good condition.
- Take particular care when using the product on damp surfaces.

6358E ALUMINIUM SHOWER SEAT

- Fit the backrest tubing between the 2 side uprights with armrests using four 6x50mm screws.
- Fit the 2 cross struts supporting the seat (one in front, the other at the rear) between the two side uprights with armrests using four 4x25mm screws. The pinched ends of the cross struts should face forwards and down. Warning: before installing the second cross strut, fit the 2 aluminium "Vs" that support the toilet bowl into the notches provided with the point of the "Vs" facing outwards.
- Fit the footrest onto the two side supports with the « U » opening to the rear and the outside of the curve facing downwards using four 4x60mm screws.
- Tighten all the screws securely.
- Clip the plastic seat onto the rear cross strut.

9680E AND 9681E ADJUSTABLE SHOWER SEAT AND STOOL

- Install the first part of the base tubing onto the lower part of the seat with the curved side uppermost and screw it in place.
- Screw on the second part of the base tubing.
- Adjust the height of the legs using the spring buttons. All the legs should be set to the same height.
- For the 9681E, insert the chair back into the rear of the seat and clip it in place.

H207 AND H209 SHOWER SEAT AND STOOL

- For the H209, install the plastic chair back on the rear tubing and check that it is correctly engaged.

O100 HEIGHT ADJUSTMENT KIT FOR H207 AND H209

- Remove the 4 plastic caps from the legs.
- Insert the extension pieces provided with the kit and insert the 4 adjustment clips into the holes at the desired height.
- Before use, check that the fixing clips are firm, the whole assembly is solid and the 4 legs are all set to the same height.

H203 FOLDING SHOWER SEAT

- Remove the protective cover from the fixing block.
- Determine the height of the seat (height below the knees with a person seated).
- Mark a horizontal line on the wall according to the height desired for the seat (please see the table below – all measurements are in cm):

Seat height	Line height	Seat height	Line height
46	34	52	40
48	36	54	42
50	38		

- Mark out a drilling template 120 mm x 108 mm (l x h).
- Drill each of the four corners and fix the block with four 4x40 screws and plugs. Replace the protective cover.
- Unfold the shower seat and adjust the two legs to the same height using the clips.

- Check that the seat is horizontal and the assembly is correctly fixed to the wall before use.
- If necessary, lubricate and protect the hinges with Vaseline.

MAINTENANCE

Clean regularly with a standard household cleaner. The seat on the 6358E can be put through an autoclave (1 hour at 80°). For disinfection in hospital environments, please follow the hygiene specialist's recommendations.

USER'S MAXIMUM WEIGHT

Please see the table at the end of this document.

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