
Invacare® Accessories

Invacare® Areste (L110), Invacare® Statio (L871)


en	Back Support / Bed Cradle User Manual	2
de	Rückenstütze/Bettbügel Gebrauchsanweisung	2
es	Soporte para la espalda/armazón para cama Manual del usuario	3
fr	Support dorsal/Arceau de lit Manuel d'utilisation	3
it	Supporto per schiena / archetto del letto Manuale d'uso	4
nl	Rugsteun/dekenboog Gebruiksaanwijzing	4
pt	Suporte dorsal/Armação para cama Manual de utilização	5
bg	Опора за гърба/Рамка за легло Ръководство	5

L110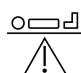

 = 100 kg

L871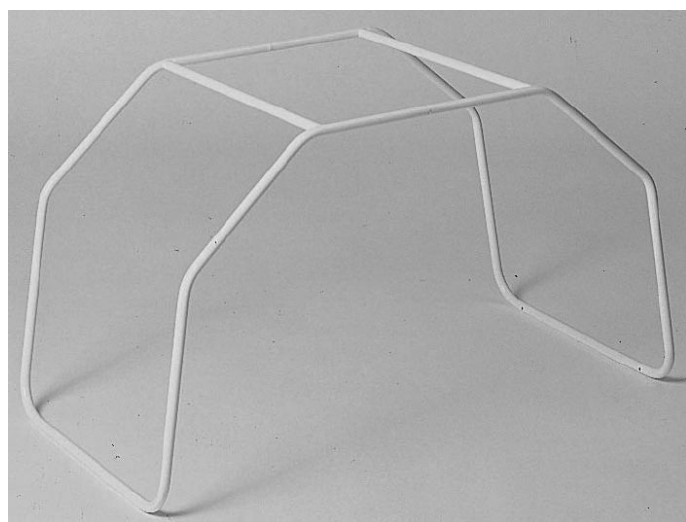

 = 25 kg

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

Safety

CAUTION!

- Check that the product has been delivered in perfect condition before you use it.
- If you are unsure about usage, adjustment or maintenance of the product, contact your Invacare distributor.
- Do not exceed the maximum load, shown on the product label.
- Do not use or store the product outdoors.
- Periodically check stability and rigidity of the product and, where applicable, that the screws are tightened.

Cleaning and disinfection

- Remove any dirt or dust using a soft, damp cloth.
- Use soap and water to remove stubborn stains.
- Dry with a soft cloth.
- For disinfection in hospital environments, follow the hygiene specialist's recommendations.

Warranty information

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located at the end of this manual.

Areste (L110) back support

Usage

The angle of the back support can be adjusted from 45° to 60° or placed in a flat position.

1. Place the back support on the mattress at the head end of the bed.
2. To adjust the angle, move the rack from one slot to another.

Technical Data

Maximum user weight	100 kg
Frame material	Steel tube
Fabric material	Nylon

Statio (L871) bed cradle

Usage

The bed cradle can be placed on the bed to alleviate pressure from the duvet on the legs or feet.

1. Position the cradle at the foot end above the patient.
2. Arrange the duvet over the cradle.

Technical data

Safe working load	25 kg
Frame material	Steel tube

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© 2017 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare AB