
# Invacare® H2080

en	Shower chair User Manual .....	2
de	Duschstuhl Gebrauchsanweisung .....	4
fr	Chaise de douche Manuel d'utilisation .....	6
it	Sedia da doccia Manuale d'uso .....	8
es	Silla de ducha Manual del usuario .....	10
pt	Cadeira de chuveiro Manual de utilização .....	12
nl	Douchestoel Gebruiksaanwijzing .....	14
sv	Duschstol Bruksanvisning .....	16
fi	Suihkutuoli Käyttöohje .....	18

## Images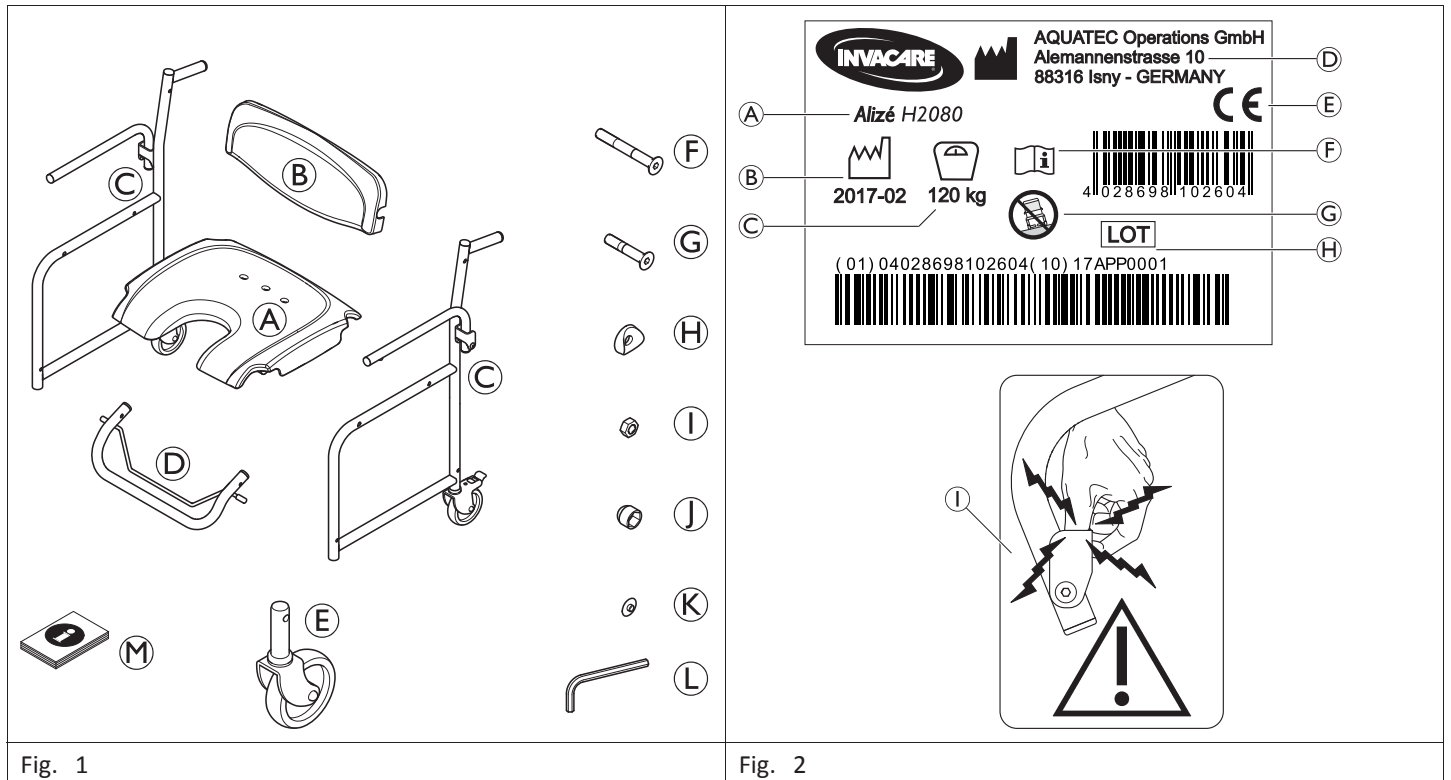

This manual MUST be given to the user of the product.  
 BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

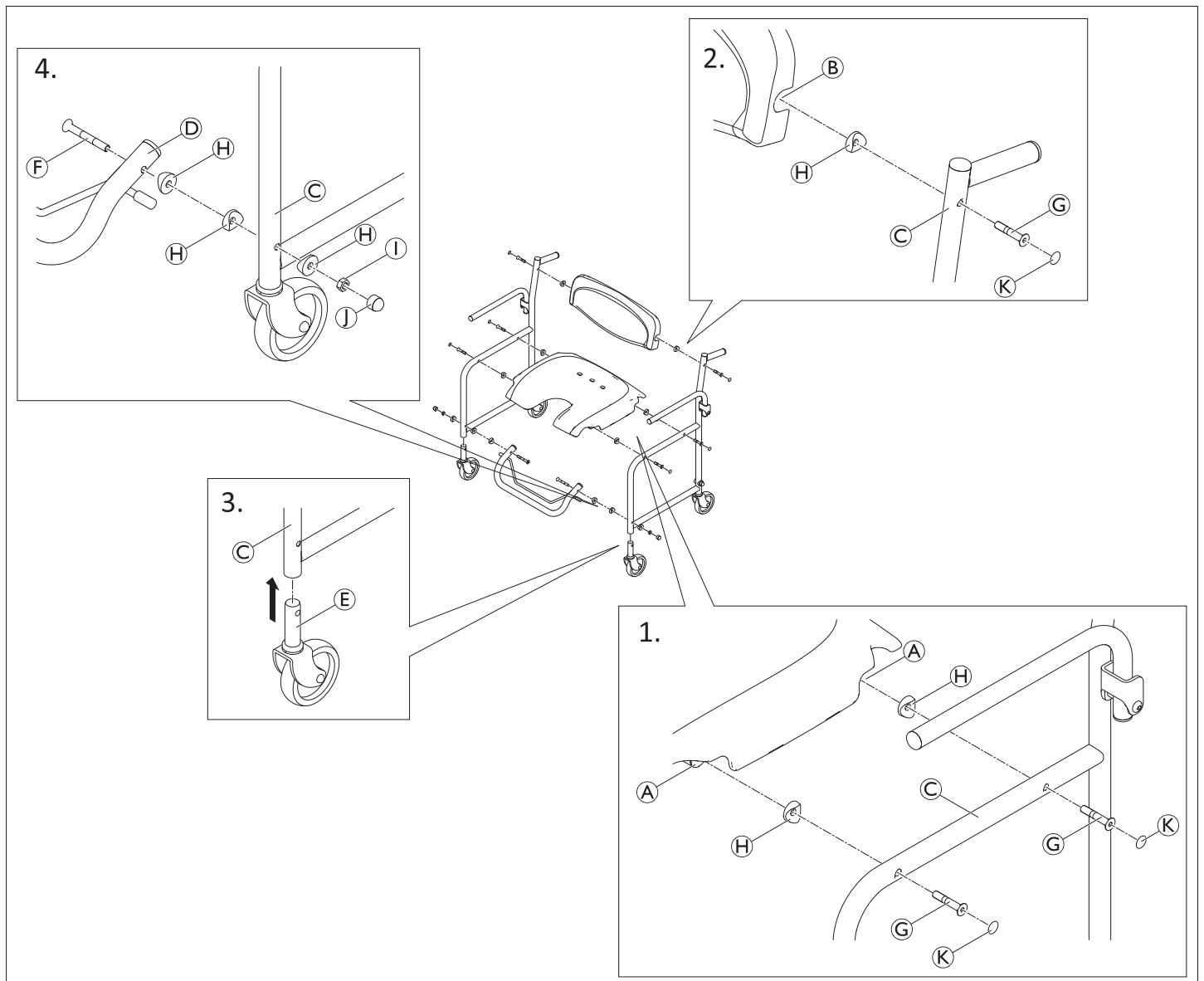
Fig. 3

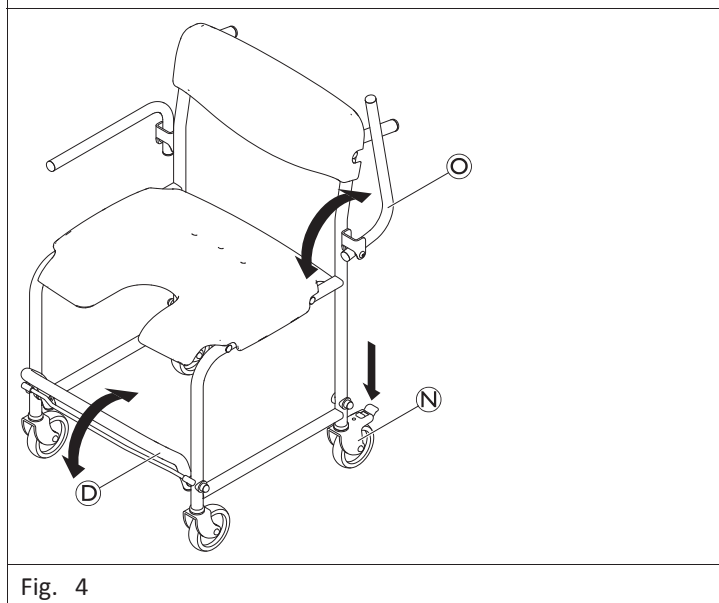
Fig. 4

## General

### Important note:

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For

addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

## Intended use

The product is intended exclusively as an aid for showering and for use in the bathroom by adult people who have:

- Difficulty standing or keeping their balance
- Weak legs or lower limb disability
- Medical conditions that impair their balance

The product is intended for homecare use only and not for intensive use in institution. Any other use is prohibited.

Operating temperature: 10 - 40 °C

Max. user weight: 120 kg

## Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

## Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

## Scope of delivery (Fig. 1)

(A)	Seat	(H)	Saddle spacer (10x)
(B)	Backrest	(I)	Nut (M6) (2x)
(C)	Frame with armrest and rear castor with wheel lock (left and right)	(J)	Covering cap for nut (2x)
(D)	Footrest	(K)	Covering cap for screw (6x)
(E)	Front castor (2x)	(L)	Allen key
(F)	Screw (M6x65) (2x)	(M)	User manual
(G)	Screw (M6x50) (6x)		

## Service life

The expected service life of this product is three years when used in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

## Safety

### Safety information

#### WARNING!

##### Risk of injury!

- Do not use if defective.
- Keep product away from open flames.
- Do not place smoldering cigarettes on the product.

#### WARNING!

##### Risk of falling!

- Do not stand on this product.
- Do not use as a climbing aid.

#### WARNING!

##### Risk of burning your skin!

- The product components can heat up when exposed to external sources of heat.
- Do not expose the product to strong sunlight before use.

#### WARNING!

##### Risk of injury!

- Only use spare parts and accessories intended for this product.

## Symbols on the product

The identification label is located on the inner side of the rear frame. Refer to Fig. 2.

(A)	Product designation
(B)	Date of manufacture
(C)	Maximum user weight
(D)	Manufacturer address
(E)	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.
(F)	Read user manual
(G)	For use on even surfaces only
(H)	Lot number
(I)	Warning label: Risk of entrapment when folding the armrest (located on the inner side of the rear frame under the armrest on each side)

## Assembly

#### CAUTION!

##### Risk of entrapment

- Be careful during assembly.

#### CAUTION!

- Follow the correct direction of mounting the saddle spacers.

Perform each of the following steps on both sides of the shower chair. Refer to Fig. 3.

1. Mount both tubes of the seat (A) to the frame (C) using screws (G) and saddle spacers (H). Cover the screw heads with caps (K).
2. Mount the tube of the backrest (B) to the frame (C) using screws (G) and saddle spacers (H). Cover the screw heads with caps (K).
3. Mount the front castor (E) into the frame tube (C).
4. Mount the footrest (D) to the frame (C) using screws (F), saddle spacers (H) and nuts (I). Cover the nuts with caps (I).

## Using the shower chair (Fig. 4)

### Getting into and out of the shower chair

#### WARNING!

##### Risk of injury

- Only use the shower chair on flat surfaces.
- Before sitting down on or standing up from the shower chair engage the wheel locks of both the rear castors.

#### CAUTION!

##### Risk of trapping your fingers!

- Do not keep your hand on the handle when raising the armrest. When lowering the armrests, ensure that your fingers are away from the hinges to avoid catching them.

1. Place the shower chair in a suitable position.
2. Engage both wheel locks on the rear castors (N).
3. Fold up the footrest (D).
4. Fold up the armrest (C) for easier transfer if necessary.
5. Transfer to the seat of the shower chair.
6. Fold down the armrest (C).
7. Fold down the footrest (D) and place your feet on it.

### Moving the shower chair

1. Hold the shower chair at the push handles.
2. Release both wheel locks on the rear castors.
3. Move the shower chair to the desired position.
4. Engage both wheel locks on the rear castors.

## Maintenance and periodic inspection

The product is maintenance-free, provided that the cleaning and safety instructions outlined in this user manual are adhered to.

#### WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled. Make sure that all screws are securely tightened.
- Regularly clean and disinfect the product.

## Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

1. Clean the product regularly.
2. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

#### IMPORTANT!

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local infection control policy.
  - Clean by hand.
  - Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.
  - Max. cleaning temperature 85 °C for 3 minutes.

## Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before re-use, clean and disinfect the product thoroughly (see chapter Cleaning and disinfection, page 4), check the product for damages and ensure a proper mounting.

## Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

## Materials

Seat / Back	ABS
Frame	Steel, powder-coated

All product components have a corrosion protection and do not contain natural rubber latex.

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- Dépliez l'accoudoir ©.
Invacare Poirier SAS
Invacare Limited
Aquatec Operations GmbH