Aquatec® ORCA / ORCA F / ORCA XL

en	Bath lift	
	User Manual	3
de	Badewannenlifter	
	Gebrauchsanweisung	31
fr	Élévateur de bain	
	Manuel d'utilisation	63
it	Sollevatore da vasca	
	Manuale d'uso	97
es	Elevador de bañera	
	Manual del usuario	129
pt	Elevador de banheira	
	Manual de utilização	159
nl	Badlift	
	Gebruiksaanwijzing.	191
da	Badekarslift	
	Brugsanvisning	221
no	Badekarheis	
	Bruksanvisning	249
sv	Badkarslyft	
	Bruksanvisning	277
cs	Vanový zvedák	
	Návod k obsluze.	305

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

AQUATEC®

Yes, you can.

©2018 Invacare Corporation

All rights reserved. Reproduction, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

1 General	4
1.1 Introduction	4
1.2 Symbols in this manual	4
1.3 Warranty	4
1.4 Compliance	5
1.4.1 Product-specific standards	5
1.5 Intended use	5
1.6 Service life	5
2 Safety	6
2.1 Safety information	6
2.2 Safety information on electromagnetic compatibility	7
2.3 Labels and symbols on the product	7
3 Setup	9
3.1 Safety information	9
3.2 Scope of delivery	9
3.3 Setting up the bath lift	10
3.4 Charging the battery	13
4 Usage	15
4.1 Safety information	15
4.2 Controlling the bath lift	15
4.3 Bathing	16
5 Transport	18
5.1 Safety information	18
5.2 Removing the bath lift	18
6 Maintenance	20
6.1 Servicing and inspection	20
6.2 Cleaning and disinfection	20

7 After Use	22
7.1 Storage	22
7.2 Re-use	22
7.3 Disposal	22
8 Troubleshooting	23
8.1 Identifying and repairing faults	23
9 Technical Data	25
9.1 Dimensions and weight	25
9.2 Electronic data	25
9.3 Materials	26
9.4 Environmental Parameters	26
10 Electromagnetic compatibility (EMC)	27
10.1 General EMC information	27
10.2 Electromagnetic emission	27
10.3 Electromagnetic Immunity	28

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Note that there may be sections in this document, which are not relevant to your product, since this document applies to all available models (on the date of printing). If not otherwise stated, each section in this document refers to all models of the product.

The models and configurations available in your country can be found in the country-specific price lists.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

For more information about the product, for example product safety notices and product recalls, contact your Invacare representative. See addresses at the end of this document.

1.2 Symbols in this manual

Symbols and signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Indicates a hazardous situation that could result in serious injury or death if it is not avoided.

CAUTION

Indicates a hazardous situation that could result in minor or slight injury if it is not avoided.

IMPORTANT

Indicates a hazardous situation that could result in damage to property if it is not avoided.

Tips and Recommendations

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.

Manufacturer of the product.

1.3 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims

can only be made through the provider from whom the appliance was obtained.

Covers and suction cups are excluded from this warranty.

1.4 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

We comply with the current environmental legislations WEEE and RoHS.

1.4.1 Product-specific standards

The product meets the requirements of the standard ISO 10535 (Hoists for the transfer of disabled persons).

For further information please contact Invacare in your country (for addresses, see end of this document).

1.5 Intended use

Bath lifts facilitate respectively allow entering and exiting the bathtub. They are intended solely as a bathing aid for people with reduced mobility. Any other use is prohibited.

Indications

- Significant reduction in function / missing functionality of the upper and/or lower extremities (such as amputations, paralysis, joint disorders, neuromuscular disorders with impairment of the extremities), when an independent entering and exiting from the bathtub respectively sitting down and getting up is no longer possible; Trunk control and basic functional capability of the extremities must still be sufficient.
- Chronic consequences of illness with functional limitations of the upper and/or lower extremities, that no longer permit an independent bathing, however when a largely independent bathing is possible by the use of the bath lift and other bathing aids for compensating the disability are not sufficient.

Contraindications

There are no contraindications associated with proper use.

1.6 Service life

The expected service life of this product is five years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

2 Safety

2.1 Safety information

WARNING!

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. Invacare product manuals are available on the internet or at your local dealer (Addresses are displayed on the back cover of this manual).
- If you are unable to understand the warnings, cautions or instructions, please contact a healthcare professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

WARNING!

Risk of injury

- Do not use if defective.
- In the event of a malfunction, contact your dealer immediately.
- Do not make any unauthorized changes or alterations to the product.
- Only use spare parts intended for this product.

WARNING!

Risk of injury

- Never stand on the bath lift to get into or out of the bathtub, do not use it for climbing or descending, as a lifting platform or for any similar purpose.

IMPORTANT!

The information contained in this document is subject to change without notice.

- Check all parts for shipping damage and test before using.

2.2 Safety information on electromagnetic compatibility

WARNING!

Risk of malfunction due to electromagnetic interference

Electromagnetic interference between this product and other electrical equipment can occur and disturb the electrical adjustment functions of this product. To prevent, reduce or eliminate such electromagnetic interference:

- Only use original cables, accessories and spare parts, to not increase electromagnetic emission or reduce electromagnetic immunity of this product.
- Do not use portable radio frequency (RF) communications equipment closer than 30 cm to any part of this product (including cables).
- Do not use this product near active high-frequency surgical equipment and the RF shielded room of a system for magnetic resonance imaging, where the intensity of electromagnetic disturbances is high.
- If disturbances occur, increase the distance between this product and the other equipment or switch it off.
- Refer to the detailed information and follow the guidance in chapter 10 *Electromagnetic compatibility (EMC)*, page 27.

WARNING!

Risk of malfunction

Electromagnetic interference may cause improper operation.

- Do not use this product adjacent to or stacked with other electrical equipment. If such use is necessary this product and the other equipment must be closely observed to verify that they are operating normally.

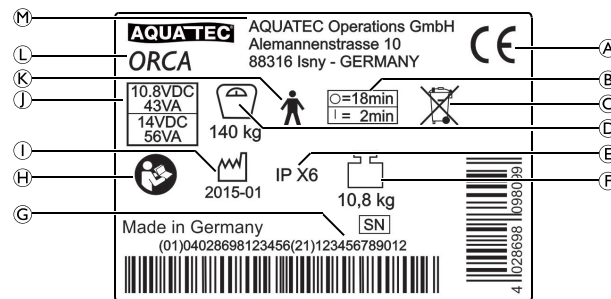
We are not aware of any risk when using products like a cardiac pacemaker.

2.3 Labels and symbols on the product

Identification label

The identification label contains important information:

Ⓐ	Conformity mark
Ⓑ	Interval of use (continuous operation of the engine is not allowed): max. 2 minutes continuous use followed by a 18 minute break
Ⓒ	Note (disposal)
Ⓓ	Maximum user weight
Ⓔ	Type of protection
Ⓕ	Product weight
Ⓖ	Serial number
Ⓗ	Read user manual
Ⓘ	Production date
Ⓢ	Nominal voltage/ rated power, see <i>9.2 Electronic data, page 25</i>
Ⓚ	Type B applied part
Ⓛ	Product designation
Ⓜ	Manufacturer

The identification label is applied to the back of the backrest.

3 Setup

3.1 Safety information

IMPORTANT!

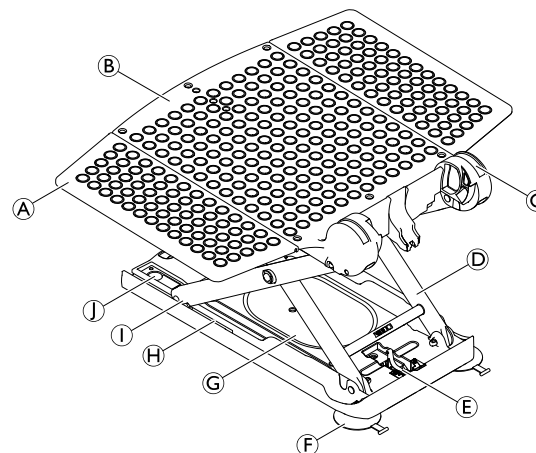
- Check the parts for transport damage before commissioning and contact the dealer if required.
- Ensure during assembly that the parts are positioned correctly with respect to one another.
- It is recommended to wash the covers before first use, see chapter 6.2 *Cleaning and disinfection*, page 20.

3.2 Scope of delivery

This user manual refers to the models ORCA, ORCA F and ORCA XL. These Models differ in weight and the maximum user weight, see section 9.1 *Dimensions and weight*, page 25. A seat- and backrest cover is optionally available for all three models. The backrest cannot be lowered on model ORCA F.

The following components are included within the scope of delivery:


Seat

(A)	Side flap
(B)	Seat plate
(C)	Hinge for backrest
(D)	Lifting scissors
(E)	Spindle foot retainer
(F)	Rear suction cups (two tabs with thread)
(G)	Base plate
(H)	Guide for the sliding shoe


①	Sliding shoe
①	Front suction cups (one tab with head)

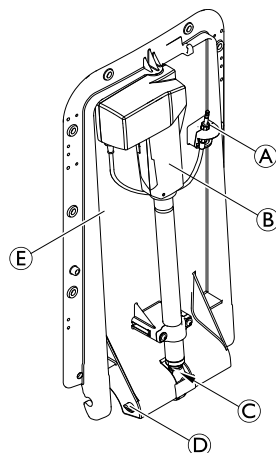Backrest

Ⓐ	Jack plug
Ⓑ	Drive
Ⓒ	Spindle foot
Ⓓ	Locking pin
Ⓔ	Plate, backrest

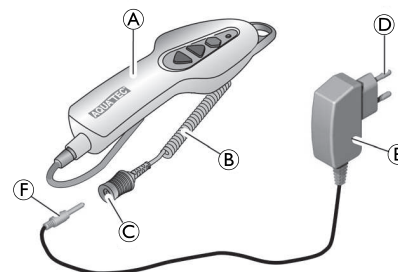Hand control and charger

Ⓐ	Hand control with battery
Ⓑ	Hand control spiral cable
Ⓒ	Jack socket of hand control
Ⓓ	Mains plug
Ⓔ	Charger
Ⓕ	Jack plug of charger

3.3 Setting up the bath lift

WARNING!


Risk of slipping

The bath lift can slip if the suction cups do not have a good hold.

- Clean the bathtub and the suction cups with a damp cloth before setting up the bath lift.
- Do not change the position of the bath lift in the bathtub once it is set up.

**WARNING!****Risk of injury and damage!**

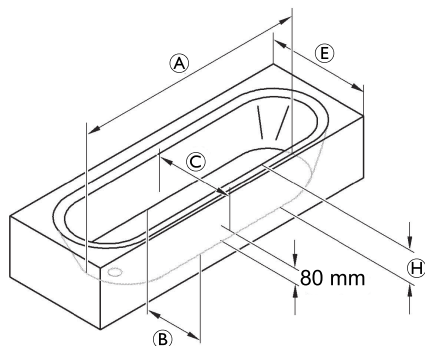- Only set up the bath lift in bathtubs that comply with the specifications of the following table.
- If a lateral overflow is present in the bath tub, use the special side flaps (available as an accessory).


Ⓐ	Tub length, bottom	≥ 800 mm
Ⓑ	Tub width, bottom (inside)	≥ 280 mm
Ⓒ	Tub width, at 80 mm height (inside)	≥ 430 mm


Ⓔ	Tub width (outside)	580 - 690 mm (with special side flaps) 700 - 740 mm (with standard side flaps) 750 - 1000 mm (with special side flaps)
Ⓕ	Tub height (inside)	≤ 420 mm (with height adapter, max. + 60 mm)

**IMPORTANT!**

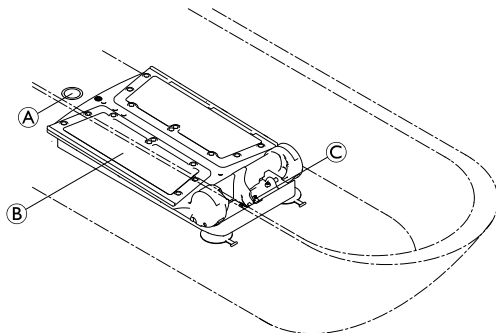
- Ensure that the suction cups are mounted properly.
- Ensure that the battery in the hand control is charged.
- Ensure that the spindle foot is fully inserted.
- When lifting bath lift components, remember to correctly assess the weights, see section 9.1 *Dimensions and weight*, page 25.
- If a handrail is mounted at the inside of the bathtub, use the side flap diverters (available as an accessory).
- Fold the side flaps outwards before folding up the backrest.

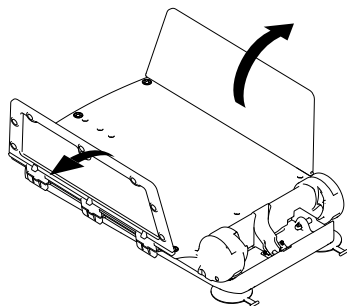1. Place the seat ② on the bottom of the empty bathtub. Ensure that the spindle foot retainer ③ is pointing towards the end of the bathtub opposite the drain ①.

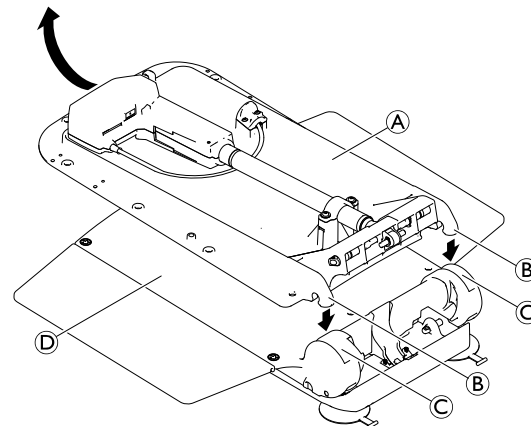2. Fold the side flaps of the seat outwards.

3. Place the backrest ① (without the hand control) onto the seat so that the hooks ② of the backrest fit into the hinges ③ of the seat plate ④.

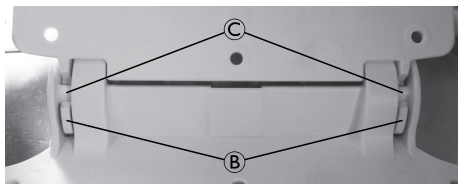
IMPORTANT!

Risk of damage


Damage to the product caused by the spindle foot slipping out.

- Make sure that the spindle is fully retracted and the spindle foot is level and not twisted, before folding up the backrest.
- Once the backrest is folded up, ensure that the spindle foot is in the spindle foot retainer.


**IMPORTANT!****Risk of damage**

Damage to the product due to incorrect assembly of the backrest.

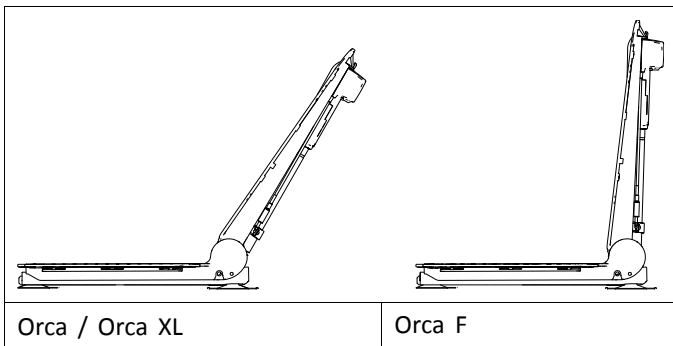- Make sure that both hooks **B** of the backrest engage into the hinges **C** of the seat plate.
- Make sure that the backrest is folded up to the end position without exerting force and noticeable resistance. The end position is indicated in the following graphic.

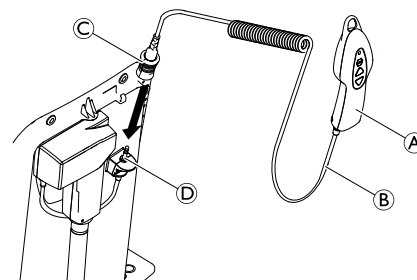4. Hold the backrest at the center of the head end and carefully fold it up to the end position.

5. Plug the jack socket with the unlocking sleeve **C** on the spiral cable **B** of the hand control **A** to the jack plug **D** until the locking mechanism engages noticeably.

**IMPORTANT!****Risk of damage**

- Make sure that the backrest doesn't touch the bath tub edge.
- Place the bath lift further forwards in the bath tub, if necessary. To place the bath lift further forwards in the bath tub lower the bath lift to the bottom, loosen the suction cups and then move the bath lift.

3.4 Charging the battery

The bath lift cannot be used when the battery is charging.

WARNING!

Risk of electric shock

- Only recharge the battery in dry rooms at room temperature (15 °C to 25 °C) and never in the bathroom.

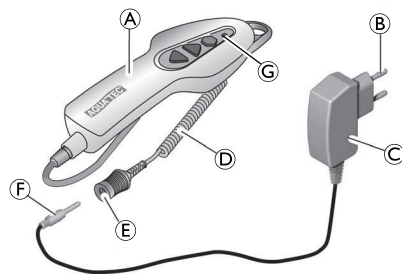
IMPORTANT!

- Only use the charger supplied to charge the battery in the hand control.
- Only connect the charger to the mains supply when the hand control is connected to the charger.
- Make sure that the voltage specifications shown on the charger correspond to those of the mains supply.

The red indicator lamp © on the hand control extinguishes when the battery is completely charged. It is not possible to „overcharge“ the battery.

3. Disconnect the charger from the power socket.
4. Pull the jack plug of the charger out of the jack socket of the hand control.

1. Insert the jack plug ⑥ of the charger ③ into the jack socket ⑤ on the spiral cable ④ of the hand control ① until it engages.
2. Insert the mains plug ② of the charger into a power socket.

4 Usage

4.1 Safety information

CAUTION!

Overloading the bath lift can lead to damage to the device or prevent lifting.

- Do not overload the bath lift and observe the specifications on the identification label and in section 9.1 *Dimensions and weight*, page 25 with regard to its load capacity.

CAUTION!

Danger of pinching fingers

- Do not insert fingers into the guide between the seat plate and the backrest.
- Do not grab under the seat plate or between the lifting scissors.

IMPORTANT!

Moisture may penetrate into a damaged hand control, which could lead to a short circuit and fire.

- Always perform a visual inspection of the product for external damage before each use.
- If the housing of the hand control is damaged or shows cracks, do not use the hand control.
- Recharge the battery completely before each use, 3.4 *Charging the battery*, page 13.

1. Refer to operating conditions in section 9.4 *Environmental Parameters*, page 26.

4.2 Controlling the bath lift

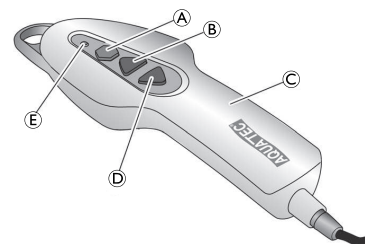
The bath lift can be moved in the following directions:

The bath lift is controlled via the hand control, either by the user or an assistant.

With a fully charged battery, the ORCA and ORCA F can perform 5 bathing operations with a load of 140 kg. The ORCA XL can perform 4 bathing operations with a load of 170 kg.

IMPORTANT!

Risk of damage

- The hand control may only be used when the backrest is properly installed and in the upright position.

! IMPORTANT!

If the battery in the hand control © is not sufficiently charged, the red indicator lamp ⑤ lights up red when the Down button ① is pressed. In this case, the lowering function of the bath lift is disabled. You can however still raise the lift.

- Recharge the battery immediately after raising the bath lift.

Raising the bath lift

! IMPORTANT!

If the backrest is lowered, it will move into the upright position first (not on model ORCA F).

1. Press the Up button ② on the hand control and keep it pressed.
The backrest moves into the upright position or the seat moves upwards.
2. Release the Up button on the hand control.
The bath lift stops in the current position.

Lowering the bath lift

! IMPORTANT!

If the seat is already lowered, the backrest lowers towards the rear (not on model ORCA F).

1. Press the Down button ① on the hand control and keep it pressed.
The seat moves downwards or the backrest tilts to the rear.
2. Release the Down button on the hand control.
The bath lift stops in the current position.

Emergency off switch

! IMPORTANT!

Only press the Emergency Off button ④ if the device does not stop immediately when the Up or Down button is released (e.g. if a button jams). The bath lift stops moving immediately and the red light integrated in the hand control lights up.

If the red indicator light switches off after the emergency off button is released, the fault has been rectified (e.g. button is released again). The device is ready to use again.

If the red indicator light stays on after the emergency off button is released, the fault has not yet been rectified. The device is not ready for use and must be repaired, e.g. in the event of damage to the electronics.

Contact your specialist dealer, if necessary.

4.3 Bathing

WARNING!

Risk of slipping

The bath lift can slip if the suction cups do not have a good hold.

- Before using the bath lift ensure that it is securely positioned and cannot tip over.
- Ensure that the bath lift is set up in the bathtub, → 3.3 *Setting up the bath lift*, page 10.

**WARNING!**

- If your condition means that you cannot take a bath on your own, only use the device with supervision.
- If operating the device without supervision, make sure to have an alarm system or telephone within reach in case of unforeseen events (e.g. malfunctions).

**CAUTION!**

- Bath oils or salts can impair the performance of the lifting scissors.
- If using such bath additives, do not exceed the recommended dose.

**IMPORTANT!**

- Run the unloaded lift once to the top and then back to the bottom (function test).
- Before using, check the temperature of the bath water.

1. Press and hold the Up button Ⓑ until the seat is level with the edge of the bathtub.
2. Make sure that the side flaps are level with the seat and rest flat on the edge of the bathtub.
3. Fill the bath with water.
4. Sit on the side flap nearest to you as you would on a chair.
5. Slide backwards until you are sitting in the middle of the seat.
6. Turn yourself into the bathing position, by lifting one leg and then the other over the edge of the bathtub.

7. Once you are sitting correctly, press and hold the Down button Ⓒ to lower the seat into the bathtub.

Once the seat has reached the lowest position, it is possible to recline the backrest by continuing to hold the Down button Ⓒ. This produces a more comfortable bathing position and allows you to immerse yourself deeper in the bath water (not on model ORCA F).

8. Keep the Down button pressed until the backrest has reached the desired position.

To get out of the bathtub, following these steps in reverse order.

9. To raise the backrest or the bath lift, press and hold the Up button Ⓑ until the seat is at the same level with the bathtub edge.

**IMPORTANT!**

- Disconnect the hand control from the device when the bath lift is not in use, see section 5.2 *Removing the bath lift, page 18, step 3.*

5 Transport

5.1 Safety information

CAUTION!

The backrest is not locked on the seat when it is folded and can come apart.

- Do not remove the suction cups when transporting the bath lift.
- Remove the hand control from the backrest for transport.
- Refer to the storage and shipping conditions in section 9.4 *Environmental Parameters*, page 26.

1. Remove the hand control.
2. Disassemble the bath lift into two sections for transport (seat and backrest), → 5.2 *Removing the bath lift*, page 18.

5.2 Removing the bath lift

1. Let the water out of the bathtub.
2. Lower the bath lift using the Down button until the backrest stops in the lowered end position, → 4.2 *Controlling the bath lift*, page 15.

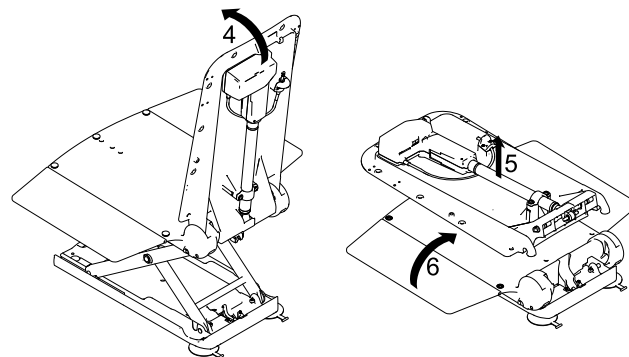
3. Unlock the jack socket on the spiral cable of the hand control by pushing up the unlocking sleeve and pull it off the jack plug.

IMPORTANT!


Risk of damage

- When folding the backrest down, ensure that the spiral cable of the hand control does not become trapped.
- Do not fold the backrest down if water is in the bathtub.

4. Hold the backrest at the top end with one hand and fold it forwards to the end position.
5. Lift the backrest up off the seat and out of the bathtub.

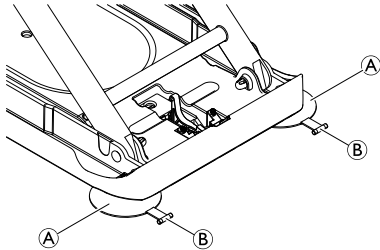6. Fold the side flaps of the seat inwards.

**IMPORTANT!**

- Release both rear suction cups simultaneously.

7. Release the rear suction cup Ⓐ by pulling up on its tab Ⓑ.
8. Lift the rear of the seat further. This releases the front suction cups.

**IMPORTANT!**

- If the height adapters have been installed (optional), you also have to pull the tabs of the front suction cups to release them.

9. Lift the seat out of the bathtub.
10. Carefully store all components of the bath lift.

6 Maintenance

6.1 Servicing and inspection

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998 (LOLER), require any equipment that is used in the workplace (e.g. by a paid carer) to lift a load be subject to safety inspection on a six monthly basis. Please refer to the HSE web site for guidance www.hse.gov.uk. Link to HSE information document on LOLER in Health and Social Care: <http://www.hse.gov.uk/pubns/hsis4.pdf>

If the cleaning and safety notices are observed, the product will be maintenance-free.

IMPORTANT!

The hand control may only be opened by the manufacturer of the bath lift.

1. Check the product regularly for damage and to ensure that it is securely assembled.
2. In the event of damage and/or necessary repairs, contact the dealer immediately.
3. The bath lift must be checked by an authorized dealer before every re-use. It is recommended to check the bath lift after 24 months at the latest (in accordance with EN ISO 10535).
4. If the bath lift is not used for a prolonged period, charge the battery completely and then raise and lower the bath lift 3 times without load. Repeat this process every 3 months. This helps to maintain the service life of the battery.

6.2 Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT!

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.

- For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.
- Clean the product regularly by hand.
- Do not use abrasive cleaning agents.

Cleaning the seat and backrest

CAUTION!

Danger of pinching fingers

- Be especially careful when cleaning in the area of the lifting scissors, as they may move and cause injury.

IMPORTANT!

Risk of damage

- Never use a high-pressure cleaner to clean the backrest.

1. Wash the product using a cloth or a brush.
2. Rinse the product with warm water.
3. Dry the product with a cloth.
4. Lightly grease the guide for the sliding shoe (Fig. Chap. 3.2, ) using Vaseline.

Cleaning the covers (optional)

It is recommended to wash the covers before first use. For hygienic reasons it is recommended to clean the covers regularly and exchange the covers yearly.

1. Unbutton the covers from the seat and backrest.
2. Wash the covers in a washing machine at max. 60 °C using a mild detergent.

IMPORTANT!

Risk of damage

- Washing at high temperatures can cause damage or shrinkage to the covers.
- Do not dry covers in the tumble dryer.

Cleaning the charger and hand control

WARNING!

Risk of electric shock

- Always disconnect the plug from the power socket before cleaning the charger.
- Never insert objects made of conductive material (e.g. knitting needles, metal pins) into the jack socket of the hand control.
- Never use a wet or damp cloth to clean the charger.

7 After Use

7.1 Storage

IMPORTANT!

Risk of damage

- Do not store the product near heat sources.
- Do not expose the product to direct sunlight.
- Refer to the storage and shipping conditions in section 9.4 *Environmental Parameters*, page 26.

7.2 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (→ 6.2 *Cleaning and disinfection*, page 20) and check the safety functions, and reset if necessary.

7.3 Disposal

The disposal and recycling of used products and packaging must be conducted in accordance with the applicable local regulations.

WARNING!

- Never throw the hand control into a fire or store it near naked flames.

Duty to inform in accordance with German Battery Directive, BattG:

This product contains a rechargeable lithium-ion battery. Discharged batteries or those which can no longer be

recharged must not be disposed of in domestic waste. Old batteries may contain hazardous substances which could be a danger to the environment and public health.

Please return such batteries to your retailer or to your municipal collection point. Returning them is free of charge and is required by law.

Please only throw discharged batteries into the containers provided and mark the poles of lithium batteries.

8 Troubleshooting

8.1 Identifying and repairing faults

The following table provides information on malfunctions and their possible causes. If you cannot correct the malfunction using the remedies named, please contact your dealer directly.

Fault	Possible cause	Remedy
No motor noise audible/lift does not move.	Jack socket of the hand control is not engaged correctly.	Engage the jack socket, → <i>3.3 Setting up the bath lift, page 10.</i>
	Battery is flat.	Recharge the battery completely, → <i>3.4 Charging the battery, page 13.</i>
	Contacts are dirty.	Clean the plug on the backrest, → contact your dealer, if necessary.
	Cable is crushed/damaged.	Replace the hand control, → contact your dealer, if necessary.
The lift can be raised but not lowered.	Low-voltage protection is activated.	Recharge the battery completely, → <i>3.4 Charging the battery, page 13.</i>
The lift stops while being raised.	Excessive load.	Observe maximum load, → <i>9.1 Dimensions and weight, page 25.</i>
The bath lift is not adhering securely to the bathtub.	Suction cups are old or damaged.	Replace the suction cups, → contact your dealer, if necessary.
Battery is not charging and the red indicator light flashes during charging	Faulty electronics.	Have your authorized dealer check the hand control.
	Faulty battery.	Replace the hand control.

Fault	Possible cause	Remedy
	Faulty charger.	Have your authorized dealer check the charger.
	Battery is too hot.	Do not charge the hand control near heat sources, e.g. heating elements, direct sunlight. Charging continues automatically once the temperature drops to the permissible limits.

9 Technical Data

9.1 Dimensions and weight

Total length at lowest position	920 mm
Total length at highest position	650 mm
Total height at highest position	1075 mm
Seat length	590 mm
Seat width with side flaps folded out	715 mm
Seat width with side flaps folded in	390 mm
Seat height at lowest position	60 mm
Seat height at highest position	420 mm
Backrest width	360 mm
Minimum angle of backrest	8°
Base plate width	295 mm
Max. load (ORCA / ORCA F) Max. load (ORCA XL)	140 kg 170 kg
Total weight without hand control (ORCA) (ORCA F) (ORCA XL)	10,8 kg 11,4 kg 11,6 kg
Seat weight (ORCA) (ORCA F) (ORCA XL)	6,6 kg 7,2 kg 7,4 kg
Backrest weight (ORCA / ORCA F / ORCA XL)	4,2 kg

Hand control weight	0,4 kg
Charger weight	0,1 kg

9.2 Electronic data

Activation force for the buttons of the hand control	< 5 N
Hand control operating voltage ¹⁾	10.8 V DC / 14 V DC
Rated input power ¹⁾	43 VA / 56 VA
Rated current	4 A
Battery type	Lithium-ion
Charger input	100 - 240 V AC, 50 - 60 Hz
Charger output	24 V DC, 135 mA, 3 VA
Charge time from low-voltage warning	approx. 4.5 h
Type of protection, drive	IP X6 W ²⁾
Type of protection, hand control	IP 67 ³⁾
Type of protection, entire device	IP X6 ⁴⁾
Sound power level	52 dB(A)

The actuator does not switch off when product is overloaded. Do not overload the bath lift to avoid damages.

1) 10,8 V DC / 43 VA valid when using hand controls with voltage 10.8 V, indicated on the back of the hand control.
14 V DC / 56 VA valid when using hand controls with voltage 14 V, indicated on the back of the hand control.

2) Protected against the penetration of water in the event of strong jets of water, tested under stipulated weather conditions.

3) Dust-proof and protected against the penetration of water during temporary immersion.

4) Protected against the penetration of water in the event of strong jets of water.

Ambient temperature	0 - 40 °C
Relative humidity	30 % - 75 %, non-condensing
Atmospheric pressure	795 - 1060 hPa

9.3 Materials

Backrest	PPGF
Seatplate	PPGF
Covers	PVC

All materials used are corrosion resistant and latex free.

9.4 Environmental Parameters

Operating conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15 % - 93 %, non-condensing
Atmospheric pressure	795 - 1060 hPa
Storage and shipping conditions	

10 Electromagnetic compatibility (EMC)

10.1 General EMC information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual.

This product has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment.

Portable and mobile RF communications equipment can affect the operation of this product.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from this product is causing the interference, run and stop running this product. If the interference with the other device operation stops, then this product is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

10.2 Electromagnetic emission

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group I	This product uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	This product is suitable for use in all establishments including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.

Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Complies	

10.3 Electromagnetic Immunity

Guidance and manufacturer's declaration

This product is intended for use in the electromagnetic environment specified below. The customer or the user of this product should assure that it is used in such an environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 8kV contact ± 2 kV, ± 4 kV, ± 8 kV, 15 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrostatic transient / burst IEC 61000-4-4	± 2 kV for power supply lines; 100 kHz repetition frequency ± 1 kV for input / output lines; 100 kHz repetition frequency	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line to line ± 2 kV line to earth	Mains power quality should be that of a typical commercial or hospital environment.

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	$< 0\% U_T$ for 0,5 cycle at 45° steps $0\% U_T$ for 1 cycles $70\% U_T$ for 25 / 30 cycles $< 5\% U_T$ for 250 / 300 cycles	<p>Mains power quality should be that of a typical commercial or hospital environment. If the user of this product requires continued operation during power mains interruptions, it is recommended that the product is powered from an un-interruptible power supply or a battery.</p> <p>U_T is the a. c. mains voltage prior to application of the test level.</p>
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	30 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Conducted RF IEC 61000-4-6	3 V $150\text{ kHz to }80\text{ Mhz}$ 6 V in ISM & amateur radio bands	<p>Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which this product is used exceeds the applicable RF compliance level above, this product should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating this product.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol: </p>

Immunity test	Test / Compliance level	Electromagnetic environment – guidance
Radiated RF IEC 61000-4-3	10 V/m 80 Mhz to 2,7 GHz 385 MHz - 5785 MHz test specifications for immunity to RF wireless communication equipment refer to table 9 of IEC 60601-1-2:2014	Portable and mobile RF communications equipment should be used no closer than 30 cm to any part of this product including cables.

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

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Invacare AB
Invacare Australia PTY. Ltd.
Aquatec Operations GmbH