Invacare® H160

en	Bath tub handle User Manual	2
de	Badewannengriff Gebrauchsanweisung	4
fr	Poignée pour baignoire Manuel d'utilisation	5
it	Maniglia per vasca da bagno Manuale d'uso	7
es	Asidero de bañera Manual del usuario	9
pt	Pega de banheira Manual de utilização	10
nl	Badkuiphandvat Gebruiksaanwijzing	12
da	Badekarsgreb Brugsanvisning	13
no	Badekarhåndtak Bruksanvisning	15
sv	Badkarshandtag Bruksanvisning	16
fi	Kylpyammeen kahva Käyttöohje	18

Images

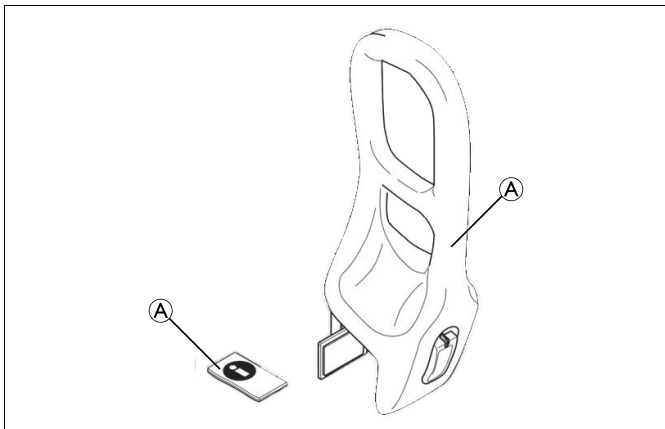
Fig. 1

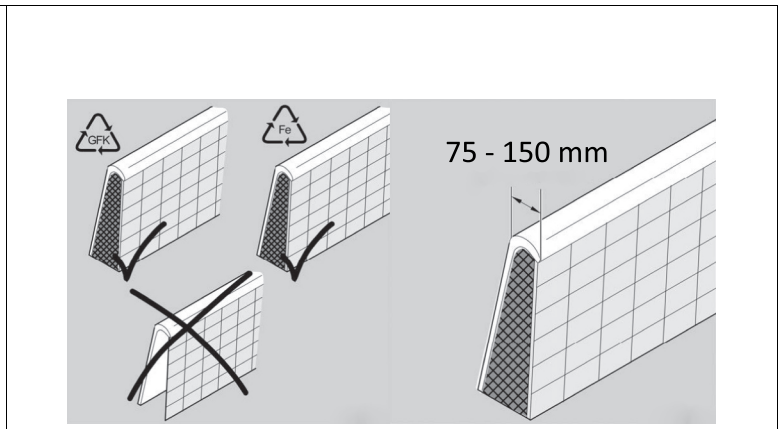
Fig. 2

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

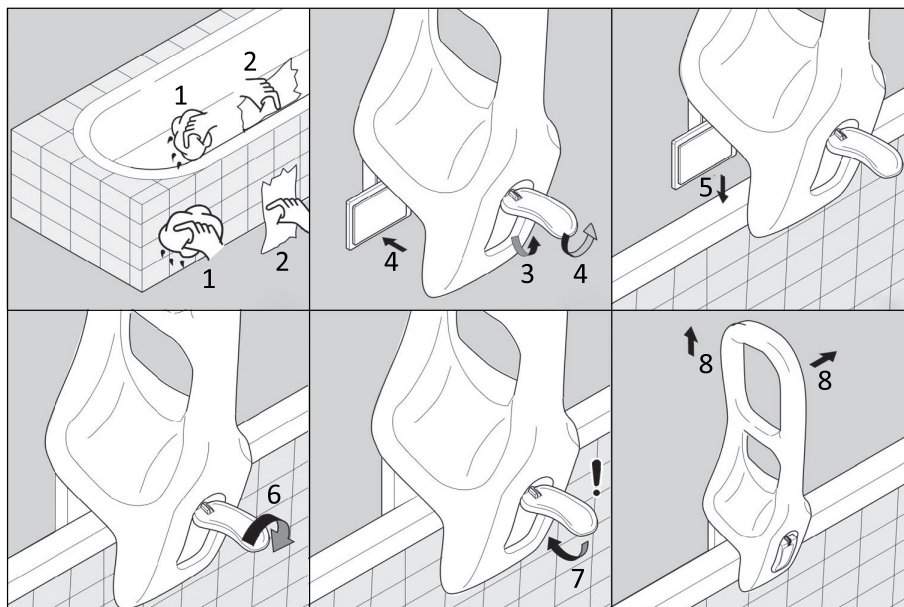
Fig. 3

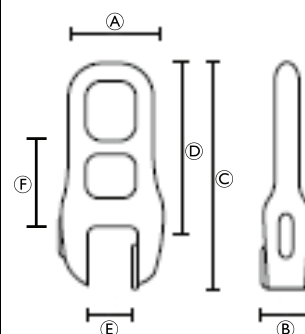
Fig. 4

Fig. 5

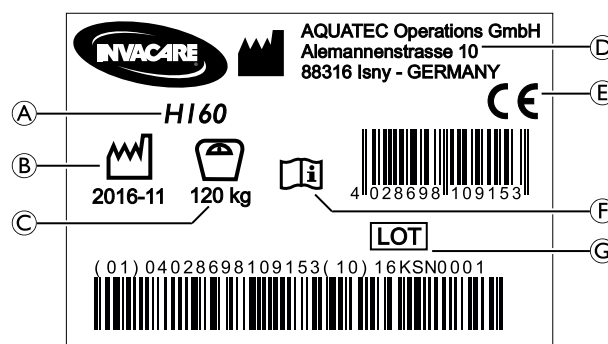
Fig. 6

en

General

Important note:

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

Intended use

The bath tub handle is designed to be mounted on bath tub side walls as a support for people with sufficient body stability to get in and out of the bath tub with additional support but not full body weight on the handle. Any other use is prohibited. The bath tub handle is not intended to bear the full body weight of the user.

Operating temperature: 10 - 40 °C

Max. user weight: 120 kg

Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see back page of this manual).

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

Scope of delivery (Fig. 1)

- Bath tub handle
- User manual

Service life

The expected service life of this product is five years when used daily in accordance with safety instructions and intended use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Safety

Safety information

**WARNING!****Risk of falling!**

- Do not use if defective.
- Do not use as sports equipment.
- Do not use as a climbing aid.

Labels and symbols on the product

Identification label (Fig. 6)

Ⓐ	Product designation
Ⓑ	Date of manufacture
Ⓒ	Maximum user weight
Ⓓ	Manufacturer address
Ⓔ	This product complies with Directive 93/42/EEC concerning medical devices. The launch date of this product is stated in the CE declaration of conformity.
Ⓕ	Read user manual
Ⓖ	Lot number

Setup and usage

Installing the bath tub handle

**WARNING!****Risk of falling!**

- Mount only to bath tubs with solid and suitable sidewalls with a wall thickness between 75 mm and 150 mm (Fig. 2).
- Do not overtighten.

Refer to figure 3:

1. Clean all mounting surfaces at the bath tub.
2. Thoroughly dry all mounting surfaces.
3. Flip the lever upwards.
4. Rotate the lever counterclockwise to open the clamp mechanism.
5. Place the bath tub handle on the bath tub sidewall.
6. Rotate the lever clockwise to close the clamp mechanism until the bath tub handle holds in place. Do not tighten too much.
7. Carefully push the lever downwards until it locks in place.
8. Check if the bath tub handle is securely fastened by pulling vertically and horizontally.

To dismount the bath tub handle reverse steps 7 to 5.

Using the bath tub handle

**WARNING!****Risk of falling or slipping!**

- Before each use, check that the bath tub handle is securely fastened and sits firmly.
- Always check that the bathroom floor is not slippery when getting into the bath tub.
- Do not jerkily tear on the bath tub handle.
- Do not expose your full body weight to the bath tub handle.

Refer to figure 5:

1522375-A

Getting into the bath tub

1. Grip the top end of the bath tub handle with one hand and carefully step into the bath tub.
2. Grip the middle part of the bath tub handle with one hand and slowly lower yourself into the bath tub.

Getting out of the bath tub

1. Grip the middle part of the bath tub handle with one hand and carefully stand up.
2. Grip the top end of the bath tub handle with one hand and carefully step out of the bath tub.

**WARNING!****Risk of falling**

- Do not use the handle to pull yourself out of the bath tub.

Maintenance

Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

**WARNING!**

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely fastened.
- Regularly clean and disinfect the product.

Cleaning and disinfection

**IMPORTANT!**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Health Care Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG139 and your local infection control policy.

- Clean by hand.
- Clean without scouring agents.
- Max. cleaning temperature 60 °C.

The product can be cleaned and disinfected using commercially available agents.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

Technical data (Fig. 4)

Ⓐ	215 mm	Ⓓ	405 mm
Ⓑ	170 mm	Ⓔ	75 - 150 mm
Ⓒ	550 mm	Ⓕ	250 mm

Max. user weight	120 kg
Product weight	2.2 kg
Material*	HDPE

*All product components are corrosion resistant and do not contain natural rubber latex.

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Invacare AB