Invacare® Classic Visco

en	Cushion User Manual	2
de	Kissen Gebrauchsanweisung	5
fr	Coussin Manuel d'utilisation	8
it	Cuscino Manuale d'uso	12
es	Cojín Manual del usuario	15
pt	Almofada Manual de utilização	19
nl	Kussen Gebruiksaanwijzing	22

This manual MUST be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

General

General information

To aid prevention of skin breakdown in a sitting position regular repositioning is recommended. Essential care, education, clinical judgment, and action based planning are fundamental factors in the prevention of skin breakdown. A range of assessment scales can be used as a formal method of assessing risk.

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions. For further information contact Invacare® in your country (addresses are on the back page of this manual).

Symbols in this user manual

In this User Manual warnings are indicated by symbols. The warning symbols are accompanied by a heading that indicates the severity of the danger.

WARNING

Indicates a potentially hazardous situation which if not avoided could result in death or serious injury.

CAUTION

Indicates a potentially hazardous situation which if not avoided could result in product damage, minor injury or both.

IMPORTANT

Indicates a hazardous situation which if not avoided could result in damage to the product.

Gives useful tips, recommendations and information for efficient, trouble-free use.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Guarantee claims can only be made through the provider from whom the appliance was obtained.

Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

The Invacare® Classic Visco cushion features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

The foam and cover used to manufacture the Invacare® Classic Visco cushion are fire safety tested and certified in accordance with EN1021 part 1 and 2.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum. We only use REACH compliant materials and components.

For further information please contact Invacare in your country (addresses are on back page of this manual).

Intended use

This cushion is intended to be used as part of an overall skin protection program of care. Designed to offer additional comfort and support to wheelchair or static seating, the Invacare® Classic Visco delivers effective pressure redistribution when in normal use. Normal use according to Invacare Ltd is when the product is placed, without additional covers or padding, between the user and the support surface. Replace the cover in case of damage. Do not use with another cover.

Product life

We estimate a life expectancy of two years for these products, provided they are used in strict accordance with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for these products does not constitute an additional warranty.

Responsibility limits

The warranty does not cover costs such as travel expenses, loss or earnings, expenses, transportation costs etc.

- Normal use
- Inadequate or incorrect use
- Incorrect assembly or installation by the buyer or third parties
- Inadequate or negligent treatment
- Use of unsuitable spare parts

Product description

Invacare® Classic Visco Cushions consist of single density, moulded Visco foam. Temperature sensitive Visco Elastic (Memory) foam allows the surface of the cushion to conform to the shape of the body. The cover consists of a two-way elastic, breathable, waterproof, and water repellent fabric.

Safety

Safety information

WARNING!

Do not use this product or any available optional equipment without first completely reading and understanding the user manual supplied. Invacare product manuals are available on your local Invacare website or at your local dealer. If you are unable to understand the warnings, cautions or instructions please contact a health care professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

**WARNING!**

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

The introduction of certain third party products between the cushion surface and the user may reduce or impede the clinical effectiveness of this product.

'Third party products' may include, but are not limited to items including under blankets, plastic sheets and sheepskins, etc. Heated over blankets must only be used in consultation with a suitably qualified health care professional, as an increase in temperature can increase the risk of developing pressure ulcers.

**WARNING!****Risk of developing pressure ulcers**

Any object between the user and the pressure reducing surface may result in the development of pressure ulcers.

- Make sure that the support surface in contact with the user is kept free from crumbs and other food debris.
- Drip cables and other foreign objects must not become trapped between the user and the pressure reducing surface of the cushion.

**IMPORTANT!**

The information contained in this document is subject to change without notice.

- Check all parts for shipping damage and test before using.
- In case of damage, do not use.
- Contact Invacare for further guidance/information.

Symbols on the product

	Refer to user manual		User weight limit*
	Do not put near flame		Avoid burning cigarette
	CE conform		Do not bleach
	Tumble dry low heat		Do not iron
	Do not pierce or cut		Machine wash temperature 90 °C
	LOT number		Manufacturing date

* Minimum / Maximum user weight as per chapter Technical Data.

Usage**Safety information****WARNING!**

Invacare advises that the patient is assessed by a trained health care professional. This is to ensure the seating solutions meet the individuals needs for skin protection and postural care.

**WARNING!**

Repositioning on a regular basis is essential. This alleviates pressure which aids in the prevention of tissue damage. This must be based on the clinical judgment of a qualified healthcare professional.

- Always consult a qualified health care professional before using the product.
- Monitor the patient frequently.

**IMPORTANT!**

In domestic settings common causes of damage include cigarette burns and the claws of pets that puncture covers, allowing fluid ingress and staining.

**IMPORTANT!**

- To prevent accidental cover damage, please ensure that sharp objects are not placed on the cover. When using patient transfer aids, care should be taken not to damage the cover. All transfer aids should be checked for sharp edges or burrs before use.

- Ensure that the cushion is not jammed or damaged by sharp edges when used on wheelchairs.

**IMPORTANT!**

The two-way stretch cover is breathable, waterproof and has a water repellent feature; this allows fluid to 'roll off' the cushion fabric, however with sustained periods of moisture the top layer of the cover may absorb fluids.

- To avoid this, wipe away any liquid as soon as it is identified.

Using the cushion

1. Remove all packaging before use.
2. Place the cushion on a wheelchair or chair. Make sure the carry handle is facing downwards and towards the rear of the seat.

Maintenance**Inspection**

Check cushion (foam and cover) for strike-through (this may include fluid ingress, stains, rips or damage) after the use of each patient and on a minimum monthly basis (depending on which occurs first) by a suitably qualified and competent person.

Check cushions

1. Unzip the cover completely.
2. Check for any staining on the interior foam.
3. Replace any damaged cover or stained foam and dispose of as per local authority procedure.

Cleaning and care

! **IMPORTANT!**
All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
– For further information on decontamination in health care environments, please refer to ‘The National Institute for Clinical Excellence’ guidelines on Infection Control www.nice.org.uk/CG139 and your local country infection control policy.

Cleaning covers

(Removal of contaminants such as dust and organic matter)

1. Remove cover for laundering.
2. Launder the cover with the recommended temperature of 90 °C using a diluted detergent solution (Instructions on label).

! **IMPORTANT!**
Washing at higher temperatures will cause shrinkage.

Drying covers

1. Hang cover from a line or bar and drip dry in a clean indoor environment.
or
Tumble dry on a low heat setting.

! **IMPORTANT!**
– Tumble dry setting must not exceed 40 °C.
– Do not tumble dry for longer than 10 minutes.
– Dry thoroughly before re-fitting to the foam.

Disinfecting covers

(Reducing the number of microorganisms)

Please contact your hygiene specialist in the event of contamination.

! **IMPORTANT!**
– Ensure that any residual detergent has been removed with clean water prior to disinfection.

Light soilage

1. Wipe down the cover with a 0.1% Chlorine Solution (1,000 ppm).
2. Rinse the cover with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

Heavy soilage

Where the cushion is badly soiled, we recommend cleaning with a dilute cleaning solution at 90 °C in the washing machine.

 Large spillages of blood should be absorbed and removed with paper towels first, followed by as above.

1. Clean up all spillages of bodily fluids and/or secretions as soon as possible using a 1% Chlorine Solution (10,000 ppm).
2. Rinse with clean water using a single use nonabrasive cloth.
3. Dry the cover thoroughly.

! **IMPORTANT!**
1% Chlorine Solution used on a regular basis can diminish the life of the cover if not rinsed and dried properly.
– Do not use granules.

 **WARNING!**
– Remove contaminated foams from use.

 **CAUTION!**
– Keep clear of open heat sources.

! **IMPORTANT!**
– Do not use phenols, alcohols, bleaches, or other abrasive materials.

Replacing covers

1. Unzip the cover and remove it carefully from the foam core.
2. Place new cover onto the foam core.
3. Close the zipper.

! **IMPORTANT!**
– Ensure that the corners of the foam core are positioned correctly into the corners of the cover.

After Use

Storage

! **IMPORTANT!**
– Store cushions in a dry environment.
– Store cushions within a protective cover.
– Store items on clean, dry surface free from sharp edges to avoid any possible damage.
– Never store other items on top of a cushion.
– Do not store cushions next to radiators or other heating devices.
– Protect cushions from direct sunlight.

1. Refer to the storage and shipping conditions in section *Environmental Parameters, page 5*.

Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used.

1. Before reuse, clean the product thoroughly, see section *Cleaning and care, page 4*.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

1. Ensure that the cushion is cleaned prior to disposal to avoid any risk of contamination.

Technical Data

General Data

Total height [mm]	75	
Total width x total depth [mm]	375 x 400	405 x 400 430 x 430 455 x 400
Nominal density range [kg/m ³]	85	
Nominal hardness range [N]	140 – 180	280 – 320

Minimum / maximum user weight [kg]	0 – 50	50 – 100
Weight of product [kg] ¹⁾	0.95	1 – 1.2

¹⁾ Weights can vary depending on size of cushion ordered, average weights used as an indication.

Materials

Foam	Polyurethane foam component
Outer cover	Polyurethane coated polyester knitted fabric
All product materials do not contain natural rubber latex.	

Environmental Parameters

Operating conditions	
Ambient temperature	10 - 35 °C
Relative humidity	30% - 75%, Non-condensing
Atmospheric pressure	70 - 106 kPa
Storage and shipping conditions	
Ambient temperature	-40 - 70 °C
Relative humidity	10% - 100%, Non-condensing
Atmospheric pressure	50 - 106 kPa

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Invacare UK Operations Limited
Invacare Limited