## Dorking Stocking Aid (Kousenhulp), (Pantyhulp)

### Doelinden

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## Dorking Stocking Aid AA4650 and AA4652 Tights Aid AA4654

## Instruction Sheet

A

B

#### Intended uses

- To compensate for a disability by helping a person put on socks, stockings, or pantyhose or tights (Model AA4654 only) without unnecessary bending at the waist or hips.

#### Instructions for installation

1. Sit on a firm surface and place feet on the floor.
2. If using Model AA4650 or AA4652, bring the two side edges of the flexible-plastic sock aid toward each other to form an open-sided tube. With the cord edge toward the floor and the open side of the tube toward the body, hold the sock aid in this position firmly between the knees. If using Model AA4654, place it on the lap with the open side up and the cord end toward the body.
3. Gather the leg of the sock, stocking, pantyhose, or tights up to the toe, keeping the thumbs inside the garment.
4. Line up the heel of the garment with the outside curvature of the sock aid and pull the garment over the end of the sock aid.
5. Continue pulling the garment onto the sock aid until it is entirely on the sock aid. Be sure that the sock aid is firmly into the toe of the garment and that the top of the garment does not extend past the edge of the sock aid where the cord is attached. Also be sure that the heel of the garment is on the curved portion of the sock aid.
6. Continue holding the cords, one on each side of the leg, while lowering the sock aid to the floor in front of the foot. Be sure the heel of the garment is down.

7. Keeping the heel of the foot firmly on the floor, slip the sock aid over the foot until the toes reach the toe of the garment. Wiggling the toes may help the garment move onto the foot. Do not pull the sock aid onto the foot with the cords at this stage.
8. Lift the heel from the floor, straighten the knee, and point the toes. Pull the cords to bring the sock aid over the heel and up the back of the calf.
9. Continue to pull the cords until the sock aid releases from the garment.
10. Proper use of Models AA4650 and AA4652 is shown in A; proper use of Model AA4654 is shown in B.

#### Care and Cleaning

The sock aid can be cleaned with mild detergent and water.

Please retain these instructions for future reference.

#### Manufactured for Patterson Medical Ltd.

Nunn Brook Road, Huthwaite,  
Sutton-in-Ashfield, Nottinghamshire,  
NG17 2HU, UK.

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